## Contasol GP rastvor za čišćenje

Rastvor za čišćenje gaspropusnih i tvrdih kontaktnih sočiva.

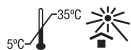
Contasol GP rastvor za čišćenje se koristi za svakodnevno čišćenje gaspropusnih i tvrdih kontaktnih sočiva.

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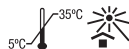
## Contasol GP Cleaner

Cleaner for gas-permeable and rigid contact lenses.

Contasol GP Cleaner is used for the daily cleaning of gas-permeable and rigid contact lenses.

The cleaning effect of Contasol GP Cleaner is guaranteed by small particles present in the solution, which eliminate deposits from the contact lens.

### INSTRUCTIONS FOR USE

1. Wash your hands.
2. Shake the bottle before each use.
3. Place the lens on the palm of your hand and apply a few drops of Contasol GP Cleaner to the lens.
4. Rub it carefully for about 10-20 seconds between your fingertips.
5. Rinse the lens thoroughly with Contasol GP Conditioner or saline solution to remove the Contasol GP Cleaner completely.
6. Fill the lens case with Contasol GP Conditioner and leave the lens there overnight or for at least 6 hours.

### PRECAUTIONS

- The product is not suitable for soft contact lenses.
- This solution should not come into contact with the eyes. If this occurs, wash them with plenty water.
- Close the bottle tightly after each use.
- Do not touch the dropper with your fingers.
- Check that the seal is intact before using the product for the first time.
- Use the product within 90 days after opening.
- Keep out of the reach and sight of children.
- Do not use this product if you are allergic to any of its ingredients.

### INGREDIENTS:

Sodium chloride, lauryl ether sulphate, silica gel, titanium dioxide, boric acid, sodium tetraborate, disodium edetate 0.1% and polyhexamethylene biguanide 0.001% in purified water.

## Contasol GP Conditioner

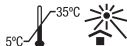
Conditioner solution for disinfecting and preserving gas-permeable and rigid contact lenses.

Contasol GP Conditioner's high wetting properties prolong comfort for contact lens wearers.

### INSTRUCTIONS FOR USE

1. Wash your hands.
2. After cleaning with Contasol GP Cleaner, fill the lens case with Contasol GP Conditioner and leave the lenses in the lens case overnight or for at least 6 hours. The contact lenses should be totally covered by the solution.
3. Remove the lenses from the lens case and rinse them briefly with Contasol GP Conditioner before putting them on.
4. If you need to take the lenses out during the day, wet them with few drops of Contasol GP Conditioner before putting them back on.

### PRECAUTIONS

- Always use a fresh solution to disinfect the contact lenses and dispose of any solution already used.
- Close the bottle tightly after each use.
- Do not touch the dropper with your fingers.
- Check that the seal is intact before using the product for the first time.
- Use the product within 90 days after opening.
- Keep out of the reach and sight of children.
- Do not use this product if you are allergic to any of its ingredients.
- In the event of eye irritation, remove the lenses, stop using the product and see a doctor or a contact lens specialist immediately.
- The lenses can be stored in the closed lens case for up to 30 days.

### INGREDIENTS

Sodium chloride, boric acid, sodium tetraborate, hydroxypropyl methylcellulose, poloxamer, disodium edetate 0.1% and polyhexamethylene biguanide 0.0004% in purified water.

Disop  
Avda. Valdelaparra, 31-A.  
28108 Madrid, España

CE 0318

STERILE A