Contasol Soft Elite

- VIŠENAMENSKI RASTVOR SA NATRIJUM HIJALURONATOM
- Čisti, uklanja proteine, dezinfikuje, vlaži i ispira
- Maksimalna hidratacija
- Produžena udobnost
- Za meka kontaktna sočiva uključujući i silikon hidrogel sočiva

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Contasol Soft Elite

- MULTIPURPOSE SOLUTION WITH SODIUM HYALURONATE
- Cleans, eliminates proteins, disinfects, wets and rinses
- Maximum hydration
- Prolonged comfort
- For soft contact lenses including silicone hydrogel

This product is a multipurpose solution that keeps your lenses clean and comfortable with its formula that contains poloxamer and sodium hyaluronate. Regular use helps to eliminate the protein deposits that accumulate on the contact lens.

INSTRUCTIONS FOR USE


1. Wash your hands before handling the lenses.
2. Remove one of the contact lenses and rinse it for a few seconds.
3. Add three or four drops of multipurpose solution to the lens and clean it by rubbing the lens gently for about 20 seconds.
4. Add enough multipurpose solution to the lens case so that when the lens is inserted it will be totally covered by the liquid, and then insert the lens you have just cleaned.
5. Remove the other contact lens and repeat the same process.
6. Leave the contact lenses in the closed case overnight or for at least six hours to obtain proper disinfection.
7. Before using the lenses again, rinse them for a few seconds with the product.

WEEKLY ENZYMATIC CLEANING

This multipurpose solution automatically

eliminates protein deposits on the lenses, making weekly enzyme cleaning unnecessary. However, if lenses become very dirty, it may be necessary to clean them with enzyme tablets.

PRECAUTIONS

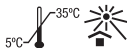- Always use fresh multipurpose solution. Discard any solution remaining in the lens case from the previous day.
- Keep the bottle tightly closed when not in use and avoid contact between the dropper and any surfaces.
- Make sure the seal is not broken before using the product for the first time. Once open, use within 90 days.
- Keep the product out of the reach of children.
- Do not use the product if you are allergic to any of its ingredients.
- In the event of persistent discomfort or irritation, remove the lenses, stop using the product, and consult your contact lens specialist.
- The lenses can be stored in the closed lens case for up to 30 days.

INGREDIENTS

Sodium chloride, boric acid, sodium tetraborate, sodium citrate, poloxamer 407, hyaluronic acid, disodium edetate 0.02% and polyhexamethylene biguanide 0.0001% in purified water.

Disop
Avda. Valdelaparra, 31-A.
28108 Madrid. España

CE 0318

STERILE A